



DISTRIBUTOR

Hunter Urology Ltd
The Threshing Barn
Woodhayes, Honiton
EX14 4TP Devon, UK

Tel. +44 (0)1404 44088
Fax. +44 (0)1404 549944

www.hunterurology.com
enquiries@hunterurology.com

MANUFACTURER

MEDICAL SERVICE GmbH
A Teleflex Company
Luisenstraße 8
D-75378 Bad Liebenzell

Tel. +49 (0)70 52/40 31 00
Fax +49 (0)70 52/40 31 20

www.medical-service.de
info@medical-service.de




SAFETYCAT® CATHETER
Feeling safe feels good.

SAFETYCAT®

OUR BASIC PRODUCT

BASIC PRODUCT

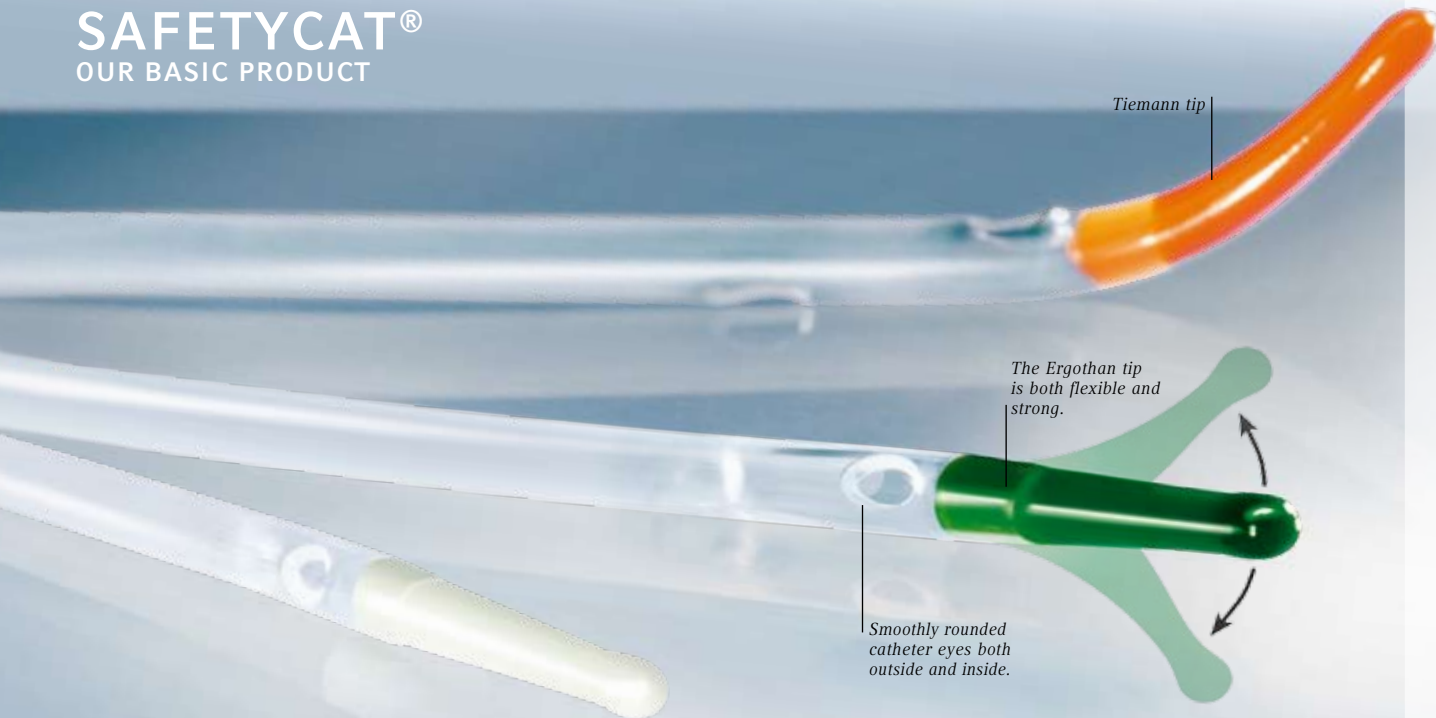

NEW ADVANCED TECHNOLOGY

At the heart of patient confidence is the feeling of safety which comes from using a comfortable catheter, that is why we developed the new SafetyCat®.

ALL-ROUND SAFETY AND COMFORT

The smoothly rounded catheter eyes protect the urethra and avoid the risk of trauma. Our unique Soft Cath Eyes technology - SCE - ensures that the eyes are smoothly rounded, both inside and out. The result - safe and comfortable self catheterisation.

THE SECRET OF SMOOTH SUCCESS

The specially designed surface of the SafetyCat® incorporates thousands of tiny pores for the storage of lubricant. This ensures the SafetyCat® is always coated with a thin layer of gel - a guarantee for easy gliding.


STRONG YET FLEXIBLE

The Ergothan tip of the SafetyCat® is both flexible and strong, enabling it to glide smoothly and easily through the urethra and sphincter.

ORDERING INFORMATION

Ch	Length	Code	Pack Size
10	Male	SCM10	30
12	Male	SCM12	30
14	Male	SCM14	30
16	Male	SCM16	30

Ch	Length	Code	Pack Size
10	Female	SCF10	30
12	Female	SCF12	30
14	Female	SCF14	30
6	Paediatric	SCP06	30
8	Paediatric	SCP08	30
10	Paediatric	SCP10	30

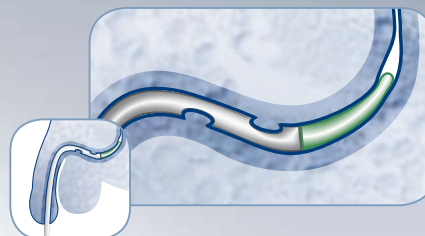

SafetyCat® is available on prescription in the United Kingdom, and on the GMS scheme in Southern Ireland.

LIQUICK® BASE


LAYING THE FOUNDATIONS FOR MODERN SAFETY

GETTING THE BASICS RIGHT IS IMPORTANT TO US.

That's why, with Liquick® Base, we're adding a new compact product to our range of catheter systems. At the heart of Liquick® Base is the SafetyCat® safety catheter. This variant has a hydrophilic-coated surface, which is quickly and easily made slippery by means of the sterile saline solution. The practical protective cover also means there's no need to handle the catheter itself, which facilitates sterile catheterisation.

THE HYDROPHILIC COATING:

When the osmolality of the hydrophilic coating combined with the sterile saline solution is equal to the osmolality of the urine (between 300 and 1500 mosmol/kg), the catheter glides easily, minimising irritation when it is inserted into the urethra. The improved drying-out time and slipperiness ensure that Liquick® Base is easy to use.

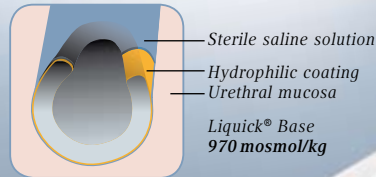

The practical protective cover means there is no need to handle the catheter itself, which facilitates sterile catheterisation.

Reservoir containing 0.9% saline solution

Available in small, compact lengths, specially designed for women.

Grip hole

SafetyCat® safety catheter with SCEs (SCE = Soft Cat Eyes) and hydrophilic coating.

ORDERING INFORMATION

Ch	Length	Code	Pack Size
10	Male	630010	60
12	Male	630012	60
14	Male	630014	60
16	Male	630016	60
8	Female	630108	60
10	Female	630110	60
12	Female	630112	60
14	Female	630114	60

Ch	Length	Code	Pack Size
8	Paediatric	630208	60
10	Paediatric	630210	60
12	Tiemann Tip	631012	60
14	Tiemann Tip	631014	60
16	Tiemann Tip	631016	60

MOBILE SL

THE GEL SYSTEM WITH STERILE LUBRICANT IN THE RESERVOIR.

SPEED OF USE AND MINIMAL VOLUME are the hallmarks of a unique catheter system: Mobile SL. In addition to the SafetyCat® catheter, the integral lubricant reservoir makes intermittent selfcatheterisation easier, safer and more hygienic.

ENVIRONMENTALLY FRIENDLY PACKAGING:

Designed without any outer packaging, Mobile SL can be kept just about anywhere. Making it less complicated for the user and better for the environment as there is significantly less waste.



Gel products:
SafetyCat® safety catheter with SCEs (SCE = Soft Cat Eyes) and microscopic indentations in the catheter surface, in which the sterile lubricant is stored.

Upper perforation

Lubricant reservoir:

Just a light touch is enough to open the gel reservoir so you can easily spread the lubricant along the catheter.

ORDERING INFORMATION

Ch	Length	Code	Pack Size
10	Male	580010G	60
12	Male	580012G	60
14	Male	580014G	60
16	Male	580016G	60
8	Female	580108G	60
10	Female	580110G	60
12	Female	580112G	60
14	Female	580114G	60

Ch	Length	Code	Pack Size
8	Paediatric	580208G	60
10	Paediatric	580210G	60
12	Tiemann Tip	581012G	60
14	Tiemann Tip	581014G	60
16	Tiemann Tip	581016G	60